DREAM OPAQUE

OPACO UNIVERSALE
E OPACO WASH
PER DREAM FRAME E METALLO

CE
0425

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DREAM OPAQUE

UNIVERSAL OPAQUE
AND WASH OPAQUE
FOR DREAM FRAME AND METAL

Product for dental use
Keep away from children
Date of specifications: 06/2019

Available packages

- DE[®] Dream Opaque Universal Opaque
One 3 g syringe available in colors:
Light (REF DO L), Medium (REF DO M),
Dark (REF DO D).
- DE[®] Dream Opaque Self Adhesive Pink Opaque
One 5 g syringe available in colors:
Pink Light (REF DOP L), Pink Dark (REF DOP D).
- DE[®] Dream Opaque Wash Opaque
One 3 g syringe (REF DOW L).

Characteristics

DE[®] Dream Opaque Universal Opaque and DE[®] Dream Opaque Wash Opaque are light-cured opaques in syringe ready to use. They have to be used after the application of UniAdhesive 1 and UniAdhesive 2 over Dream Frame carbon fiber or metal frames (see UniAdhesive application protocol).

Way of use

- Apply DE[®] Dream Opaque Wash Opaque with a stiff, short-haired brush.

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UniAdhesive application protocol

Use this protocol for the following techniques:

- DE[®] experience composite on Dream Frame carbon fiber.
- DE[®] experience composite on metal.
- 1. Sandblast the carbon fiber or the metal with 120/160 micron aluminum bixide.
- 2. Clean the carbon fiber with 94°C ethyl alcohol. Clean the metal with steamer.
- 3. Once dry, apply DE[®] UniAdhesive 1.
- 4. 2 minutes later, dry with a light blow of uncontaminated air.
- 5. Apply DE[®] UniAdhesive 2 and light cure for 2 minutes.
- 6. Apply DE[®] Dream Opaque Wash Opaque and light cure for 4 minutes.
- 7. Apply DE[®] Dream Opaque Universal Opaque and light cure for 4 minutes.
- 8. Apply DE[®] experience composite.

Useful tips

The surfaces of the framework should be cleaned of any grease and polish residue. The opaque should only be applied in very thin layers.

Working time for the materials

Depending on the lighting conditions: 1-3 minutes.

Storage

The material should be stored at 10-25°C. Close the syringe properly immediately after use and protect against the effects of direct light.

Shelf life

The maximum shelf life is printed on the label of each syringe. Do not use after the expiry date.

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Guarantee

Our technical instructions, regardless of whether they are provided verbally, in writing or during practical demonstrations, are based on our own experience and should only be considered guidelines. As our products are subject to continued development, we reserve the right to modify them. Complies with EN ISO 10477.

Note

Please supply the dentist with the above information, if this medical device is used to produce a special model. Please also consider the safety data sheets.

Side-effects

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects - even if it is doubtful that the side-effect has been caused by our product - please kindly contact us.

Contra-indications / Interactions

If a patient has known allergies against or hypersensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. The dentist should consider known interactions and crossreactions of the product with other materials already in the patient's mouth before using the product.

Via Torino, 765 - 21020 Mercallo (VA) - Italy
Phone +39 0331 969270 - Fax +39 0331 969271
www.deitalia.it

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